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Item No. 26011 200 Standard
Item No. 26011 300 Standard, XL-width

Item No. 26011 800 Extendible
Item No. 26011 900 Extendible, XL-width

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place!

Indication/Contraindication

The device for assisting in putting on compression hosiery assists persons who have limited mobility and diminished strength in the fingers and hands in putting on tight compression hosiery. The use of the device not only makes the task easier, it also helps prevent bending over and thereby minimizes the danger of falling. Polyurethane foam reinforced handles enable the user to grip and lift the device comfortably.

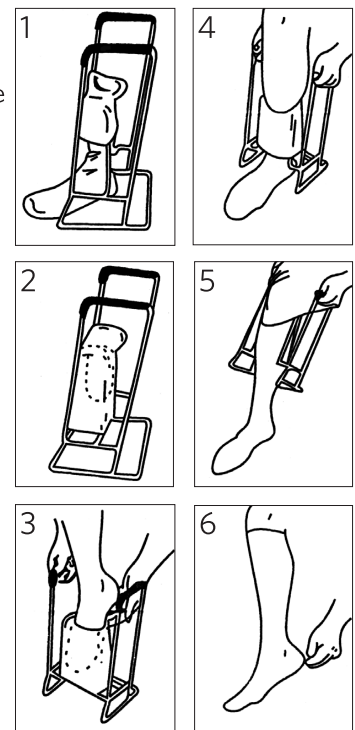
The versions with extendible handles offer the user additional support while putting on the compression stockings because of their even longer handles. When not in use the handles can be pushed back into the frame so that the dressing aid can be stored with a minimum of space requirement.

Safety Indications

- To guard against the risk of falling, use the device only while seated!
- Make sure you have maintained fingernails. To prevent damage to the knit fabric, re-move any jewellery, especially rings.
- Do not use the versions with the extendible handles to support yourself when the handles are extended. The handles would be pushed back into the frame – danger of falling!

Use

1. Sit on your bed or in a chair. The back of the device should be facing toward you. Place the compression stocking in the inner frame by stretching the stocking over the semi-circle. The heel of the stocking must face the back of the device, toward you.
2. Continue by pulling the stocking further onto the frame by pulling evenly on both sides of the stock-ing until the heel is centred on the upper edge of the semi-circle or until the heel closes below the semi-circle, depending on the size of your foot.
3. Now place the device on the floor directly in front of you. Carefully slip your foot into the raised stocking until your foot is completely touching the floor. Hint: With one hand, pull the heel of the stocking securely against the frame as you slide your toes into the stocking.
4. Use the comfortable cushioned hand grips to pull the device slowly and carefully up your leg until the stocking is above the calf. Do not attempt to overstretch the stocking – do not pull the stocking up too much.
5. Release the compression stocking completely from the device with slight jogging motions. You can then move the device away and down from your leg.
6. Ensure that the heel of the compression stocking fits your foot snugly. Adjust the stocking so that he has no wrinkles by smoothing it out with the palm of your hand.

The versions with the extendible handles offer the possibility of extending the handles to a length of 61cm. In order to extend the handles, it is best to place the dressing aid before you on the table. First extend one handle while holding it steady on the table with the other. Repeat this process with the other side so that the handles are extended to their maximum length.

Cleaning

The device may be cleaned with a moist towel using a mild household cleanser.

Material

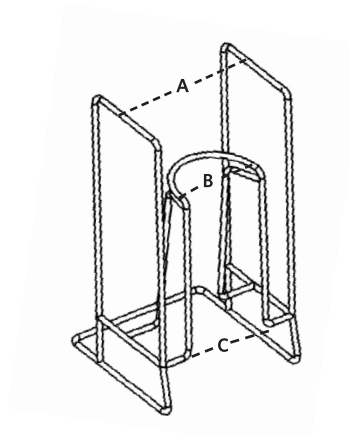
- Frame: epoxy coated metal
- Handles: Polyurethane foam

Disposal

Please obtain information from your specialist dealer about your regional waste disposal regulations.

Technical Details

- Handle height:
Standard version: 34,3 cm
Extendible version: 41-61 cm
- Distance between handles (A): 19 cm
- Width between semi-circle of inner frame (B) and between foot piece (C):
Standard and Extendible: 12,5 cm
Standard, XL-width and Extendible, XL-width: 14 cm

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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