
# User manual

Fresh

**etac**<sup>®</sup>  
Creating Possibilities

78187 - 2023-08-18 - Version 11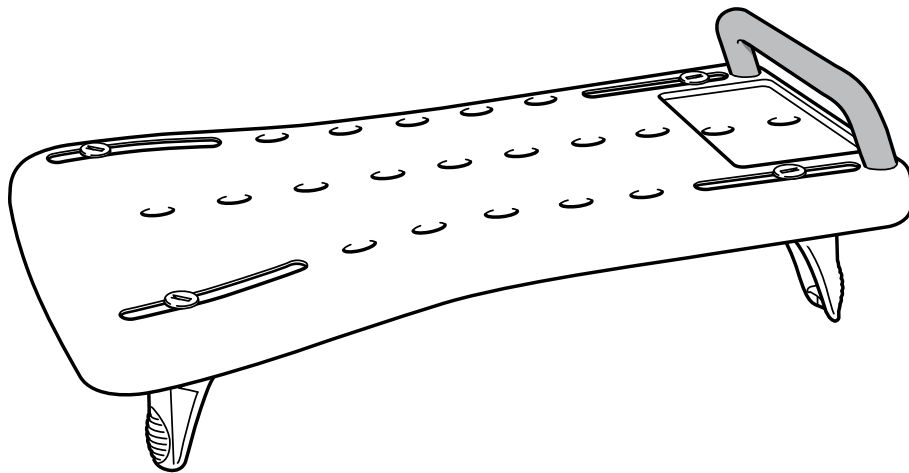


sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar

[www.etac.com](http://www.etac.com)



MD CE

sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar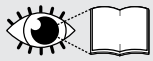

5 5 6 7 7 8 9 10 10 11 12 13 13 14 15 16 16 17 18 19 19 21


A

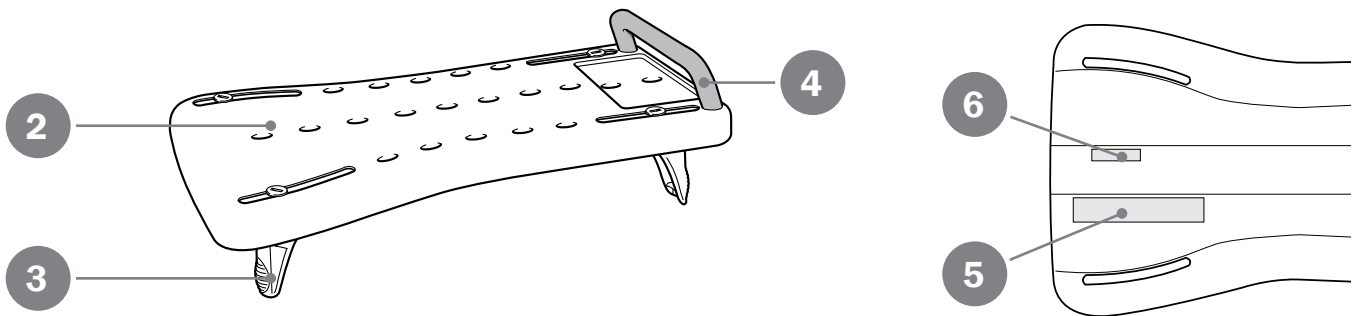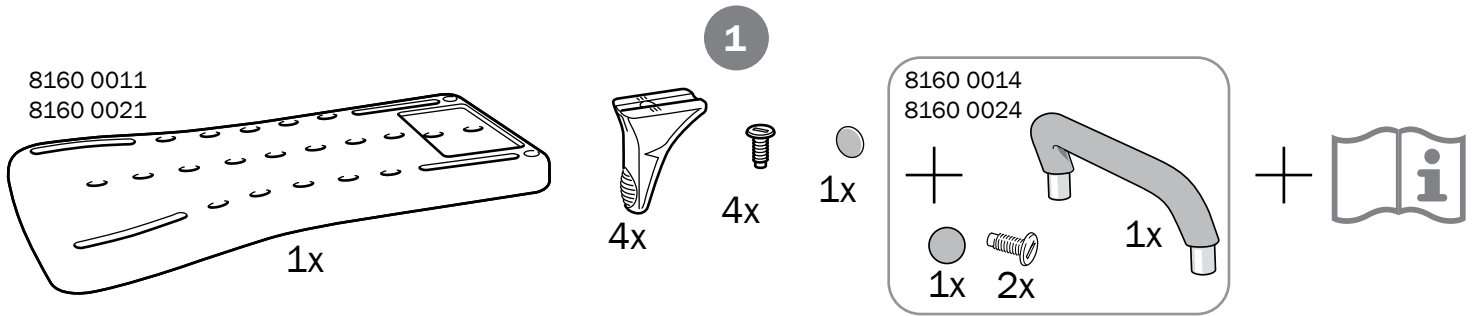B

C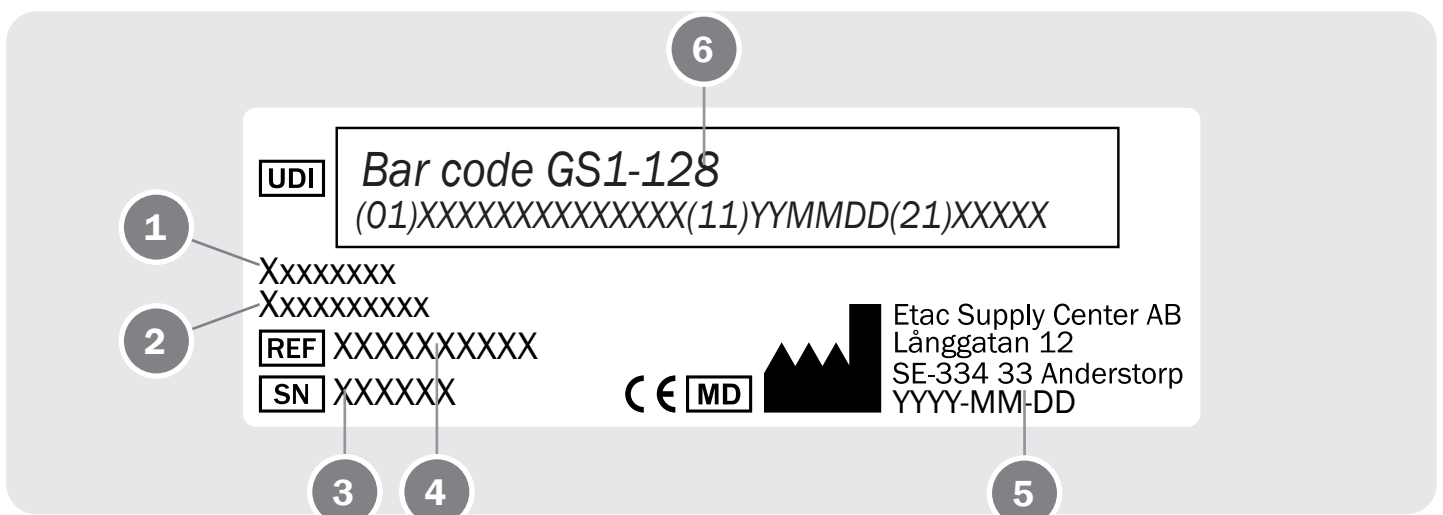

sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar

# D

	<table border="1"> <tr> <td data-bbox="794 566 1161 750"> </td> <td data-bbox="1161 566 1540 750"> <p>MAX kg lbs</p> </td> </tr> <tr> <td data-bbox="794 750 1161 918"> <p>PP</p> </td> <td data-bbox="1161 750 1540 918"> <p>150 kg</p> </td> </tr> </table>		<p>MAX kg lbs</p>	<p>PP</p>	<p>150 kg</p>
	<p>MAX kg lbs</p>				
<p>PP</p>	<p>150 kg</p>				

# E

- 1
- 2
- 3


4

max 2 cm


- 1
- 2

# F




1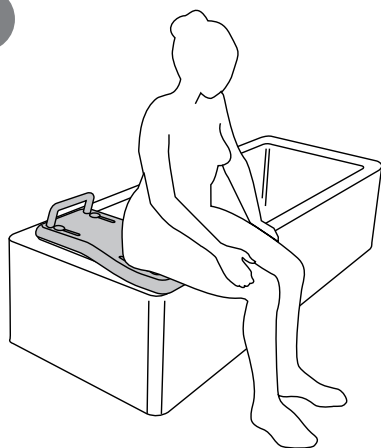


2

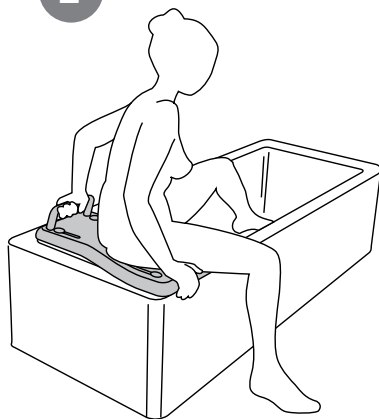3
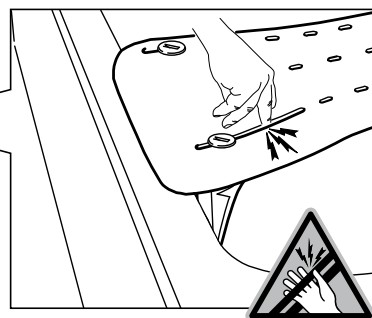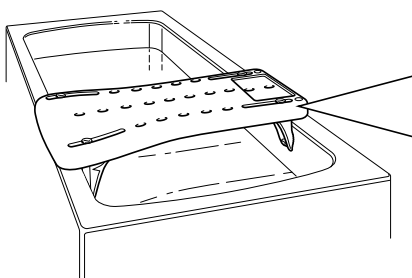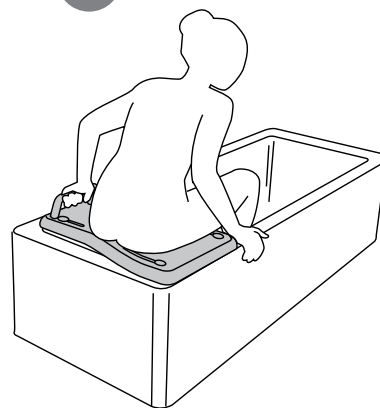
# G

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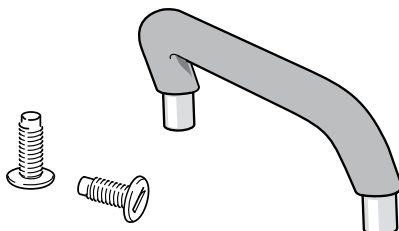### Tillbehör ..... [Figur G](#)

#### 1. Handtag

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

### Device description

Fresh is a bath board with non-slip pattern and sturdy brackets. The extra wide seating surface on one side makes it easier to get into and out of the bathtub. The seat also have drainage holes that effectively remove excess water.

### Intended purpose

Fresh (hereafter also referred to as "the device" or "the product"), is a medical device intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to facilitate personal hygiene and provide stability and support in the shower or the bathtub.

### Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more. Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

### Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

### Intended application

The device is intended for short-term and long-term use and can be applied several times per day.

The device is intended to be used in contact with intact skin.

The device is intended for refurbishment and reuse.

### Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see [www.etac.com](http://www.etac.com).

### Indications

Indication for use is disability, including, but not limited to, an individual with fluctuating balance issues, physiological and/or functional impairment, and/or a history of falls.

### Contraindications

There are no known contraindications.

### Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

Check before each use that the device is correctly assembled.

 Heated device can cause burns

 A defective device must not be used.

 Ignoring installation or assembly instructions may result in injury.

 Do not stand on the device

### Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1, SS EN 12182.

### Pre-purchase information

Pre-purchase information can be found at [www.etac.com](http://www.etac.com).

### Adverse event

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

### Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

If there is any uncertainty regarding the execution of the customization, consult Etac. Device coupled with another device may not retain either device's CE mark unless combination agreements exist. Contact Etac for current information.

### Device description ..... [Figure A](#)

1. Content of the delivery
2. Bath board
3. Edge stop
4. Handle
5. Product label
6. Item no label

### Symbols ..... [Figure B](#)

Symbols in the manual and on the device:

1. Warning, precaution or limitation.
2. Useful advice and tips.
3. Material for recycling.
4. Read instructions for use
5. Risk of crushing!
6. Maximum user weight (see technical data)
- 7-8. Cleaning (see maintenance)

### Explanation item no label: ..... [Figure C](#)

1. Product name
2. Product description
3. Serial number
4. Item number
5. Date of manufacture
6. Barcode acc. to GS1-128 GTIN-14 and serial number\*

\*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

### Technical data ..... [Figure D](#)

### Assembly ..... [Figure E](#)

### Usage ..... [Figure F](#)

 Check before each use that the device is correctly assembled.

 Risk of crushing!

### Maintenance

Clean the device with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85 °C if so required.

Constituent materials are corrosion-resistant.

5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

### Storage

The device should be stored indoors in a dry place at a temperature above 5 °C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

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**Etac Ltd.**
**Promefa AG**
**Etac Supply Center AB**