# DIAMOND DELUXE

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## Table of contents

Preface .....	10
Declaration of conformity .....	10
Service life .....	10
Disposal .....	10
Identification plate .....	10
Reuse.....	10
Warranty .....	10
Parts of the product .....	11
Intended purpose .....	12
Indication.....	12
Scope of delivery.....	12
Safety instructions.....	12
Assembly.....	13
Adjusting the brakes.....	14
Braking   Parking.....	14
Adjusting the backrest .....	14
Folding   Transporting   Stowing.....	14
Cleaning   Maintenance .....	15
Technical data.....	15

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

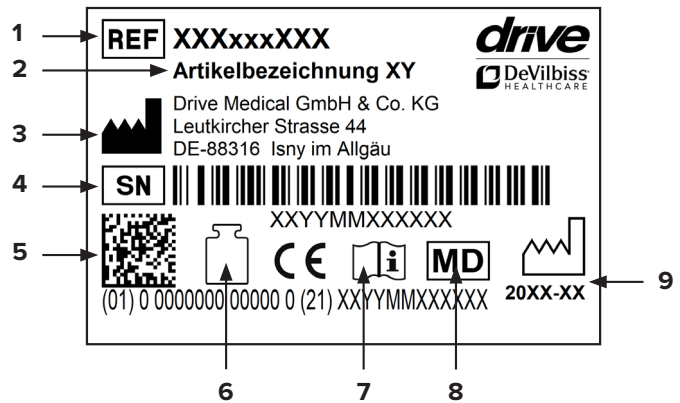1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de)

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

## Parts of the product

## DIAMOND DELUXE

- |                                  |                              |                |
|----------------------------------|------------------------------|----------------|
| 1. Backrest                      | 4. Seat                      | 7. Cane holder |
| 2. Height-adjustable push handle | 5. Rear wheel                | 8. Front wheel |
| 3. Brake                         | 6. Adjusting screw for brake | 9. Pocket      |
|                                  |                              | 10. Footrests  |

## Intended purpose

Walkers are used to support safe walking and, depending on the design, can also be used to relieve pressure. Four-wheel walking frames are products made from tubular frames which have four solid rubber wheels. These walking frames are designed such that the user always moves between the contact points or wheels. These walking frames have braking systems which operate using Bowden cables operated by handgrips. On some products there are devices or baskets to hold loads and objects, which can be transported with these frames. These walking frames can be folded for transportation and the handgrips are height-adjustable to suit different heights.

## Indication

### INDICATION:

Walking aid for outdoor and indoor use to compensate for or improve limited walking ability. Insured parties are to be provided with walking aids with the intention of compensating for or at least improving the user's limited walking ability (expansion of radius of activity or mobility). Persons with limitations of hand/arm functions require particular adjustments/designs of handles or handrests/armrests. Movable walking aids may not be suitable for persons with balance disorders; in individual cases, this should be tested out. Where appropriate, an attendant must be present. The rollator is to be used only for normal, everyday activities.

### CONTRAINDICATION:

The rollator is not suitable for use in the case of: perception disorders, severe loss of balance, inability to sit down.

## Scope of delivery



1 x	Rollator
1 x	Pocket
1 x	backrest
1 x	Cane holder
1 x	Instructions for use

## Safety instructions


- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the type label.
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake (pull the brake lever upwards).
- Leaning excessively on the handles can cause the rollator to tip backwards.
- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.
- The braking effect can be impaired by wear and tear on the tires. For this reason, check the braking effect and tires regularly.
- The rollator should NOT be carried by the seat, removable parts or the brake cables.
- Never use the footrests as assistance when getting up – risk of tipping!
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

## Assembly


Have the individual parts ready, as shown (pic. 1). Insert the rear wheels into the mounting tube (pic. 2) and secure them with the locking screws (pic. 3).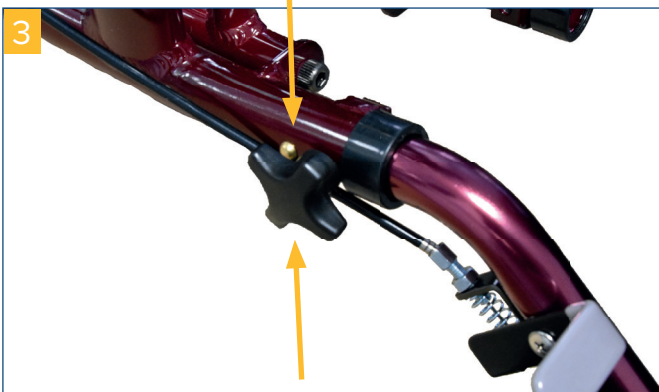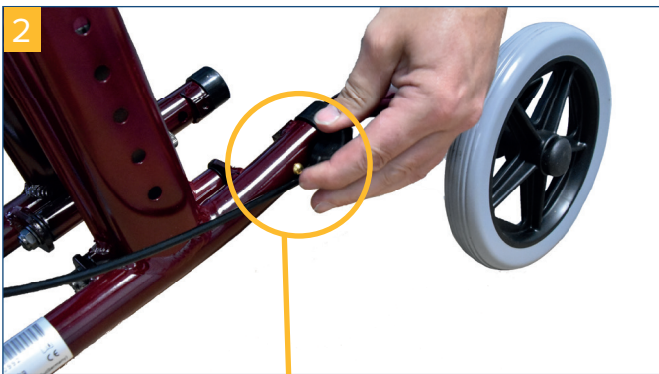




The front wheels are pushed into the holders with force until they click into place (pic. 4). To disengage the wheel, press the red button and pull the wheel out (pic. 5).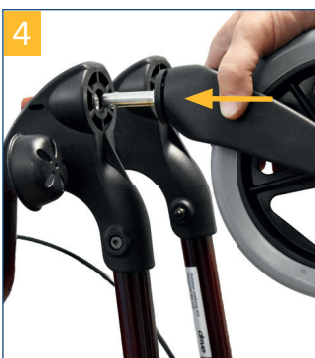

Now place the rollator (pic. 6) on its wheels (pic. 7) and press down on the sides of the seat section until you can hear and feel it click into place (pic. 8).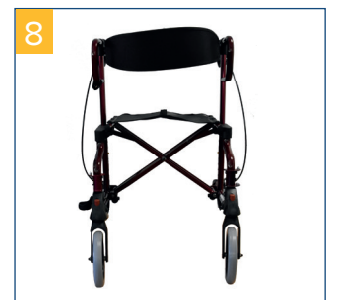

To adjust the height of the handles, press both clips and move the guide tube into the desired position (pic. 9). Hang the footrests such that they click into both pins (pic. 10,-11).

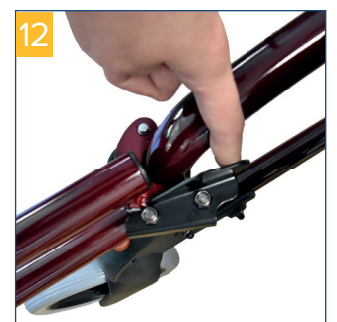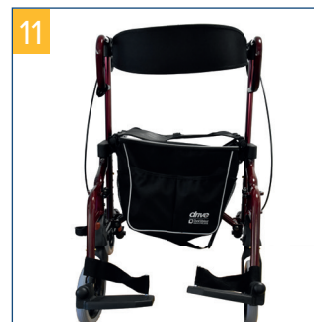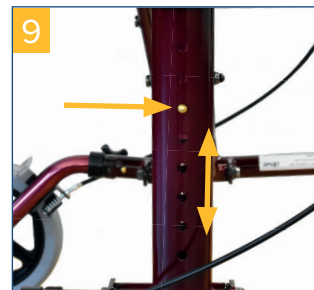
To swivel them out of the way, release the lock as shown in pic. 12. If you do not use the footrests, hang them as shown in pic. 13 such that all 3 pins are clicked into place; the footrests are then secured into position on the rollator (pic. 14).

## Adjusting the brakes

If braking is poor or too hard, the brake can be adjusted either at the brake housing above (**pic. 15**) or using the adjusting screw on the Bowden cable below (**pic. 16**) to achieve the desired braking effect.

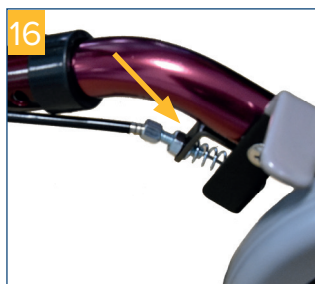The brake is correctly adjusted when the brake lever as well as the brake wires are free of play and the wheels can be rotated without grinding noises when the brake is released.

## Braking | Parking

To brake when walking with the rollator, pull both brake handles upwards with your hands (**pic. 17**). Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place (**pic. 18**)! To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position).

The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.

## Adjusting the backrest



To adjust the backrest in the respective user position, open the red locking lever on both sides (**pic. 19**), swivel the backrest into the desired position and close the locking mechanisms.

Ensure that the locking mechanism is correctly clicked into place.

## Folding | Transporting | Stowing

To fold up the rollator, pull the seat and rollator upwards! Tip rollator backwards and downwards against its direction of movement. The frame will then fold automatically (**pic. 20-22**).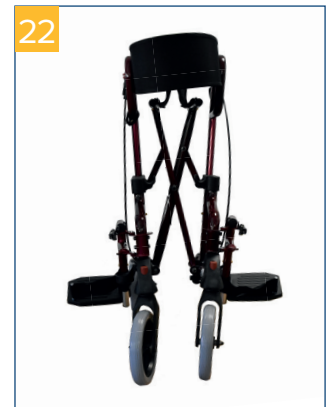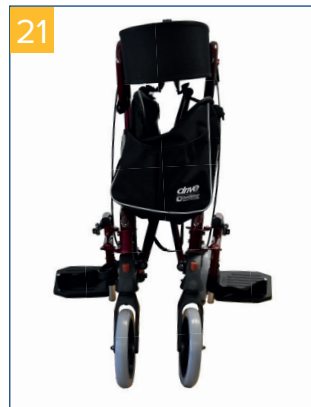

Remove accessories, if necessary!

## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.)!
- In the event of defects, contact your specialist dealer from whom you purchased this product!
- Check the condition of the tires regularly. They are "fail-safe", meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

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Drive Medical GmbH & Co. KG