Etac My-Loo

med kantstopp / with brackets / mit Klammern

1A15172.4 2022-11-01

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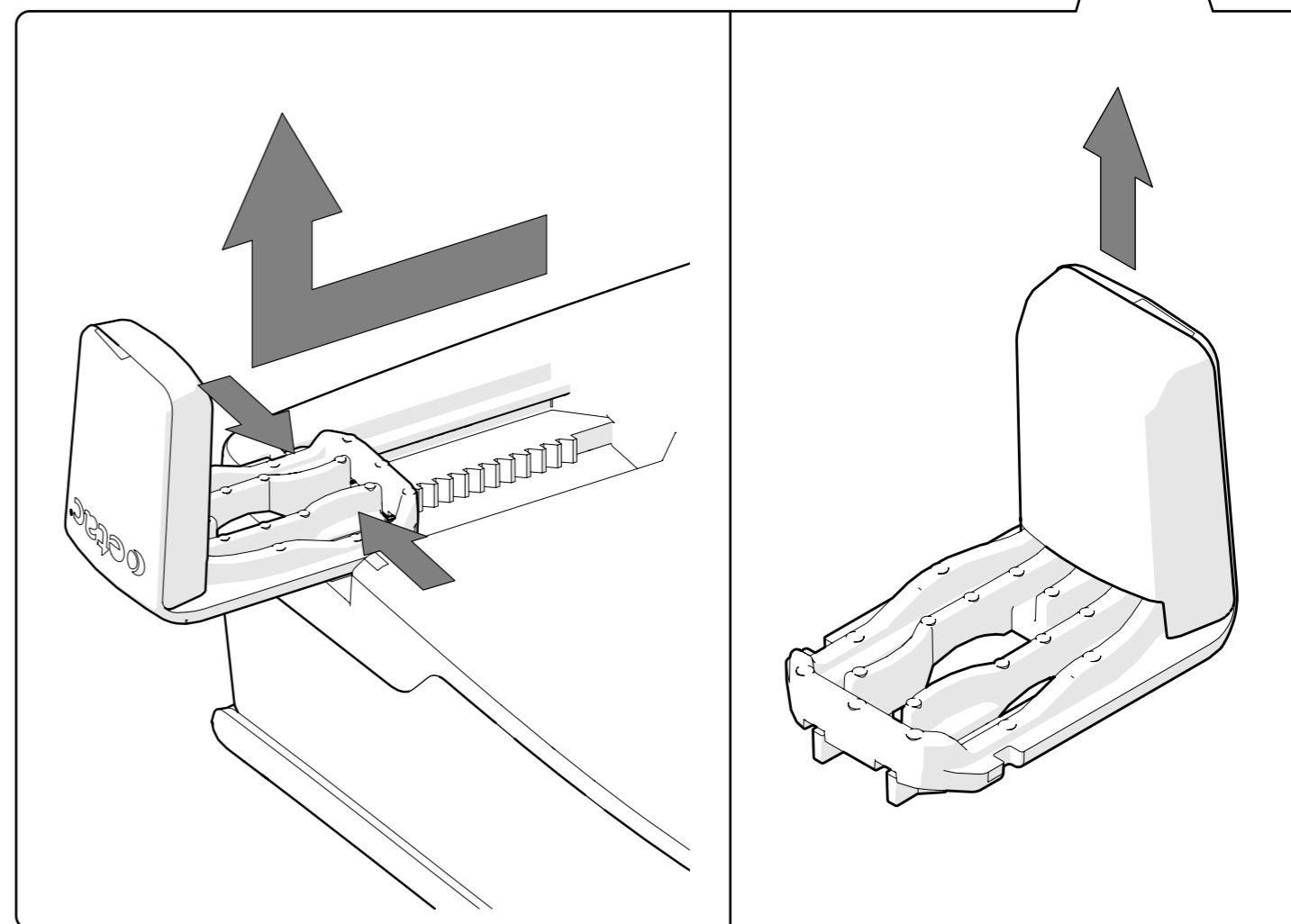
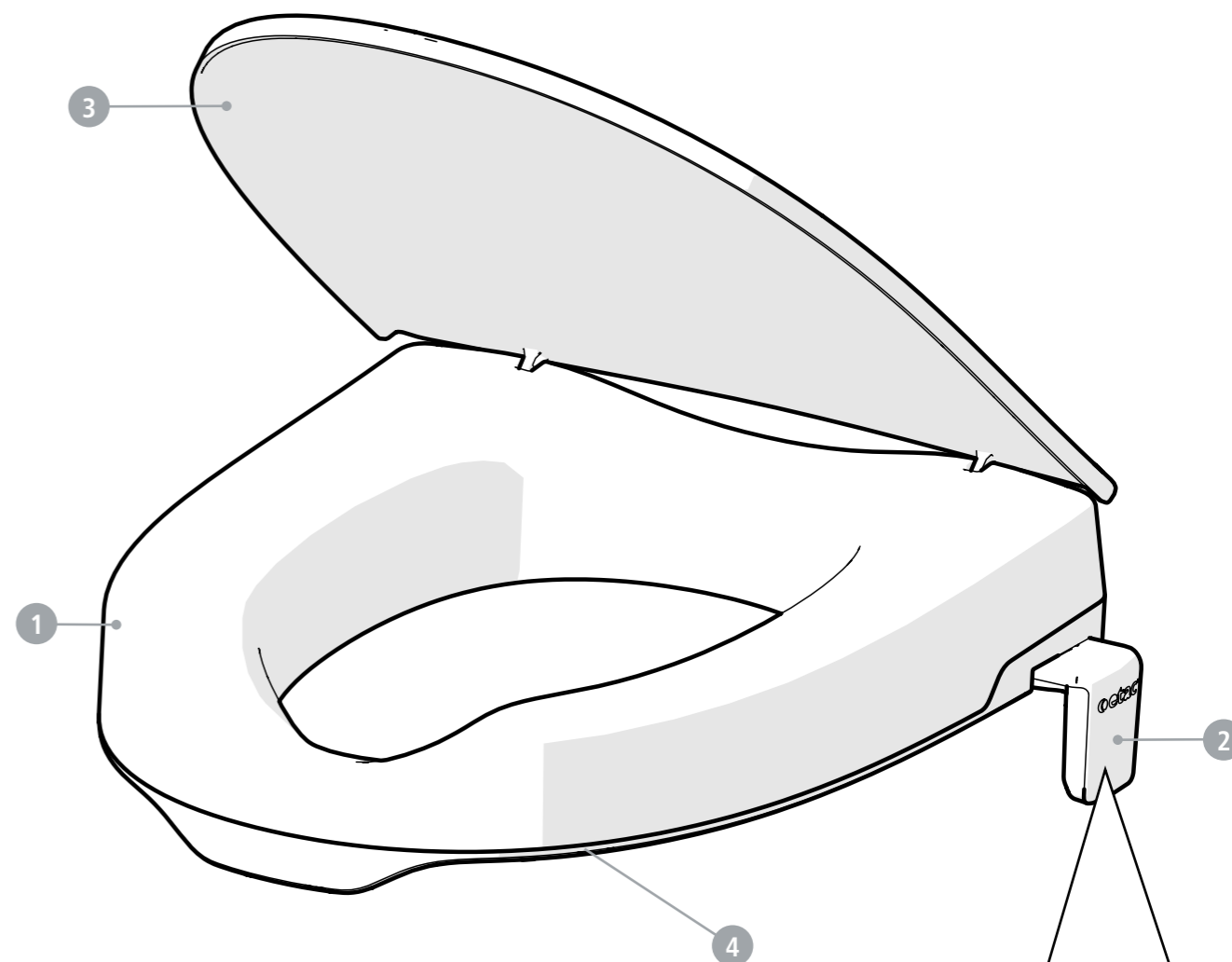
General

- The product is suitable for reuse.
- Original spare parts from Etac should always be used.
- Remember that the service life of the product is limited (see

manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

Washing

- Clean the product with normal nonabrasive cleaning agents (pH value of between 5-9). Rinse and dry. Disinfect the product with an alcohol-based disinfectant, 70%.

- The product can withstand cleaning in a sterilizer at 85 °C for 3 minutes and can be mechanically dried at 70° for 10 minutes.

1

Raiser

Check so that there are no cracks or damages.

2

Edge stops

Check so that the snap function is ok and that the edge stops can be adjusted.

Check so that there are no cracks or damages.

When cleaning in sterilizer, disassemble the edge stops and clean in upright position (see illustration) at 85 °C for 3 minutes.

3

Lid

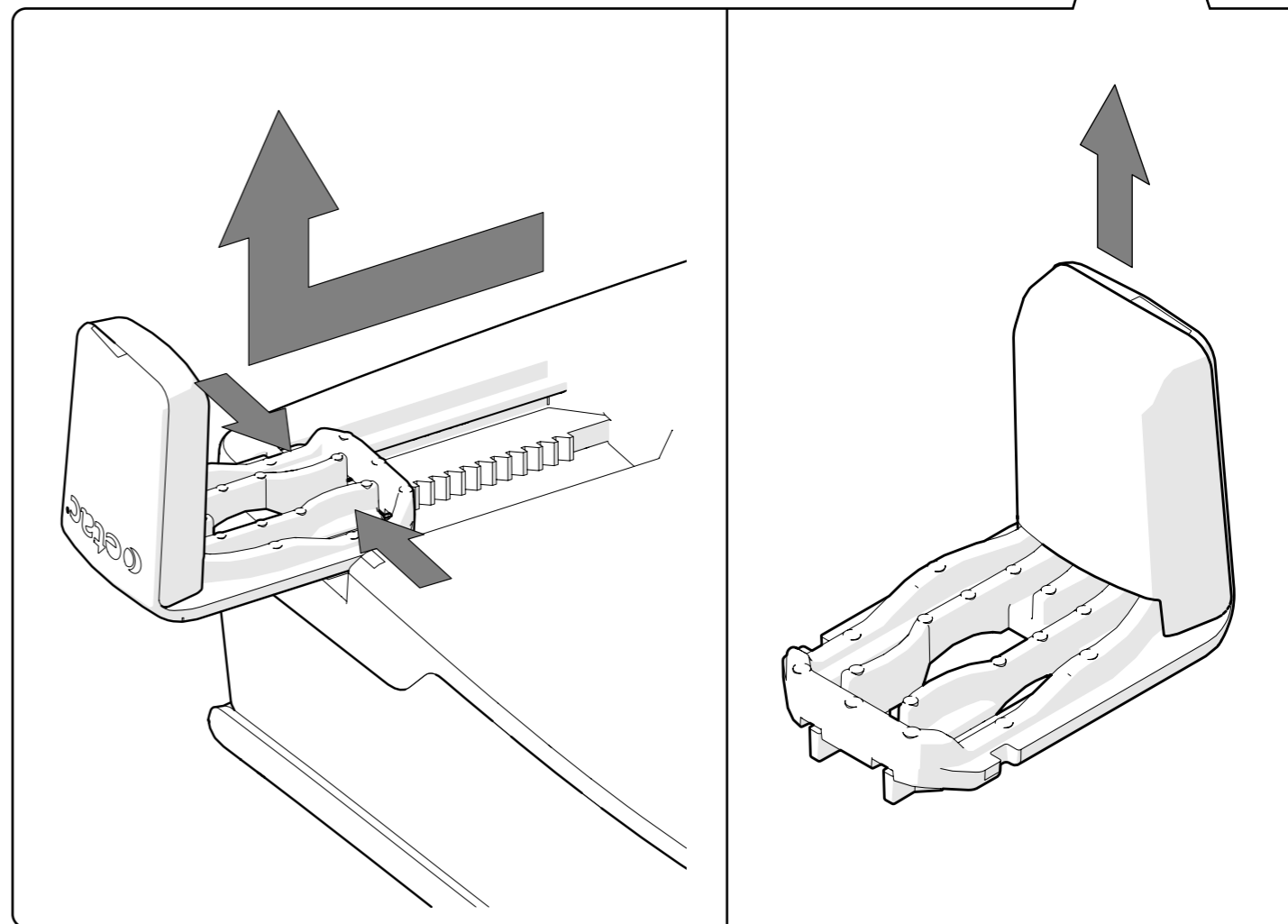
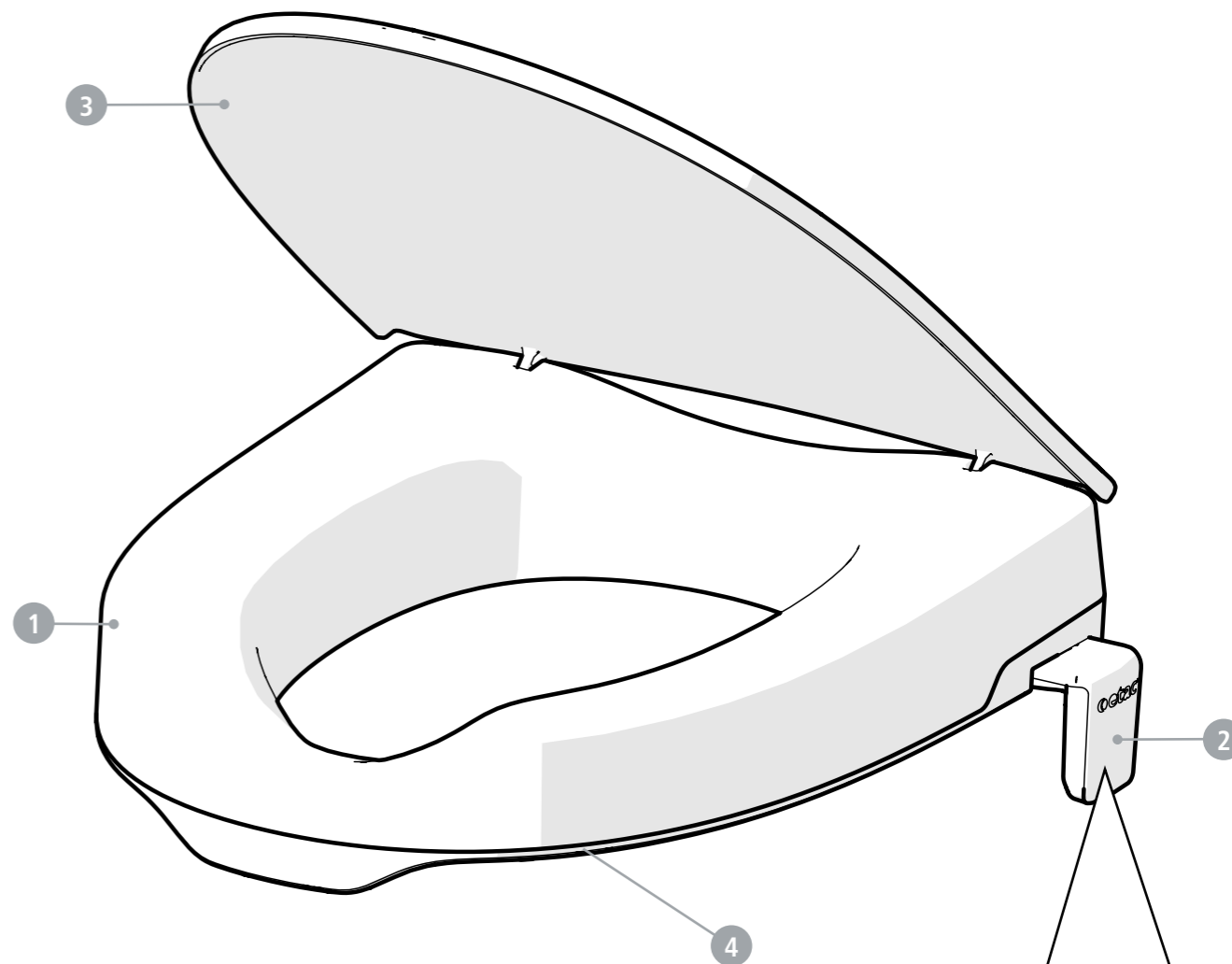
Check so that the lid is properly fixed and can be folded.

Check so that there are no cracks or damages.

4

Friction pieces

Check so that the friction pieces are properly fixed and that there are no damages.

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