

Powder

Ostomy Care



Coloplast A/S
Holtedam 1
3050 Humlebaek
Denmark

Instructions for use

23325656 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved.© 2020-12-29

Coloplast A/S
Holtedam 1
3050 Humlebaek
Denmark
Tel: +45 49 11 11 11

Intended purpose

Powder is intended for moisture absorption on intact peristomal skin.

Cautions

This non-sterile product contains raw materials of natural origin, and there is a risk that the materials carry pathogens which cannot be fully removed by even the most stringent manufacturing processes. Therefore, the product carries an increased risk of infection for vulnerable patient groups and should be used with caution for babies and infants with immature immune systems and infants receiving immune suppressive treatment.

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Special storage conditions

The powder should be stored away from direct sunlight, in a cool and dry place.

How to use**Preparation**

1. Wash and dry the skin around the stoma.

Application

2. Open the lid and aim at the intended area.
3. Shake or gently squeeze the bottle and spread the powder on the intended intact skin.
4. Spread the powder evenly using gauze.
5. Dust off excess powder before applying the ostomy baseplate or ostomy bag adhesive.
6. Close the lid tightly after use and store in a cool, dry place.

Disposal

The product should be disposed of with normal household waste or according to local guidelines. Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols

Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



Recyclable packaging